


(Box of 10)

DARVALL Cozy™

Exotics & Small Animal Blanket

For Anesthesia & Surgery and Cages: Pre-Warming or Recovery

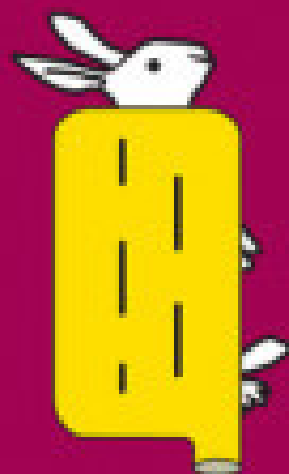

- ☐ Effective
- ☐ Thermodynamic
- ☐ Solving Hypothermia
- ☐ Rapid Response
- ☐ Safe Warming

Specifications:

Exotics & Small Animals

Blanket consumable

Inflated size:

0.6m (24")L x 0.33m (13")W

Inflation tubes 8cm(3")diam

Use with Darvall's
Kennel Door Adapter

*Designed for use with
The Darvall Cozy™ Heater

HEPA filter antimicrobial 0.2um
replace / 500 hours

Designed by Vets for Veterinary Use Only
Patents granted & pending

Guide for use:

- Compatible with most warm air heaters*
- For Surgery: Place blanket underneath with white side to the animal, as indicated on inlet port
- Inflate blanket after the animal is draped for surgery
- Recovery/ICU: Position the small animal in the cage & place the blanket over the top with white side to the animal. Alternatively place animal on blanket
- Insert heater hose nozzle into the outside of the kennel door adapter (Part 5366) & connect one of the blanket inlet ports to the inside of the adapter
- Do not occlude the inflation tube at the blanket inlet
- Follow the heater's instructions to commence warming
- Monitor the animal's temperature frequently; once it reaches 37°C (98.6°F), set heater to 38°C (100°F)
- Replace blanket if hair covered, wet or soiled

GVP



1300 838 438

sales@generalvetproducts.com.au
www.generalvetproducts.com.au

